



ETRAN (eye transfer) Instructions

- The frame is held by the communication partner and 18" in front of the user's face at eye level.
- **The user looks at the group where the letter they wish to communicate is positioned and then to the corresponding color circle of that letter.**

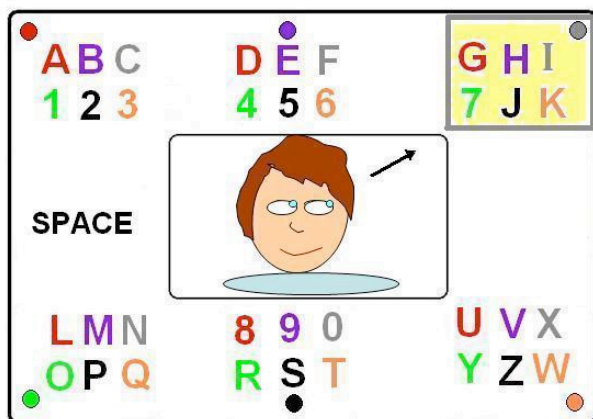

Step 1: Choose letter's GROUP

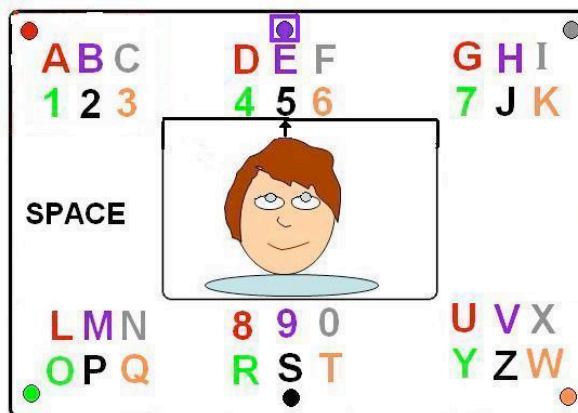

Step 2: Choose letter's COLOR

- The communication partner observes the user's eyegaze selections through the cut-out center portion and confirms (optional: writes) each letter.
 - To confirm, indicate the letter to the user; user will then indicate yes or no.
 - User communication of yes/no:
- If appropriate, partner can pre-empt the word the user is building up and confirm with user.